CLEARFIL™ AP-X

LIGHT- CURED COMPOSITE RESIN

ENGLISH INSTRUCTIONS FOR USE

GENERAL INFORMATION

CLEARFIL AP-X is a light-cured, radiopaque restorative composite resin which provides accurate colour-matching, high polishability and excellent physical properties, making it ideal for both anterior and posterior restorations. It is formulated with the optimal viscosity for easy handling and placement. CLEARFIL AP-X will give best results when used with one of CLEARFIL bonding systems, such as CLEARFIL LINER BOND 2V or CLEARFIL SE BOND. Used in combination with CLEARFIL LINER BOND 2V or CLEARFIL SE BOND, CLEARFIL AP-X will produce restorations with superb aesthetics and durability, high bond strength and less pulpal irritation.

PRECAUTIONS

- CLEARFIL AP-X should not be used with patients who have a history of hypersensitivity to methacrylate monomer.
- If CLEARFIL AP-X comes into contact with the skin or clothing, wipe off with a cotton pledget moistened with alcohol.
- In the event of resin entering the eye accidentally, flush immediately with copious amounts of water and consult a doctor.
- Avoid looking directly at the curing light when curing CLEARFIL AP-X. Wear protective glasses.
- Keep material out of reach of children.

I. INDICATIONS

- CLEARFIL AP-X is indicated for the following restorations:
1. Class I, II, V restorations of posterior teeth
 2. Class III, IV, V restorations of anterior teeth
 3. Cervical cavities or defects involving root surfaces

II. SYSTEM SHADES AND COMPONENTS

Shades

CLEARFIL AP-X is available in 14 shades; 11 shades correspond to those of the VITA® Shade Guide plus 3 shades for special needs. Select an appropriate shade by matching the shade guide to the restoration site. The XL and HO shades offer a strong, lighter white color. The HO shade is transcendent white and is suitable for sites requiring extra white color. The CL shade corresponds to A2 opaque and is suitable for cervical sites where opacity is required.

Standard shades: A2, A3, A3.5, B2, B3, C1, C2, C3

Cervical shades: A4, B4, C4, CL

Transparent shades: XL, HO

III. CLINICAL PROCEDURES

1. Shade selection
Teeth should be cleaned in case of plaque or stains. Before isolating the teeth, select an appropriate shade using the VITA® Lumin- VACUUM shade guide.
2. Cavity preparation
CARIES DETECTOR is recommended to serve as a guideline for safe caries removal. Isolate the teeth using rubber dam or cotton rolls.
3. Pulp protection
Generally there is no need for cement lining or basing.
4. Application of bonding agent
Either CLEARFIL LINER BOND 2V or CLEARFIL SE BOND is especially recommended, but all CLEARFIL bonding agents can be used. Follow the Instructions for Use.
5. Placement and light curing of CLEARFIL AP-X
 - 1) Dispensing
Dispense the necessary amount of CLEARFIL AP-X. After dispensing the resin, turn the plunger counter-clockwise a half turn to prevent excess resin from escaping. Replace the syringe cap immediately to prevent premature setting of the resin. Use the resin within five minutes of dispensing. Extended exposure to the operating light will allow the resin to cure.

2) Placement

Incremental placement and light-curing is strongly recommended especially in deep cavities and in Class II cavities.

3) Curing

Cure the resin with a dental visible light curing unit. Hold the light tip as close to the resin as possible.

shades	curing depth (mm)	
XL	20 sec	40 sec
A2, A3, A3.5, B2, B3, C1, C2	2.0	3.0
A4, B4, C3, C4, CL, HO	1.5	2.5
	1.5	2.0

6. Finishing

Contour the restoration and adjust occlusion and articulation using a fine diamond point. Polish with silicon rubber points or polishing discs.

IV. PRODUCT INFORMATION

Principle ingredients of CLEARFIL AP-X:
Silanated barium glass, silanated colloidal silica, silanated silica, Bis-GMA, TEGDMA, di-Camphorquinone

V. SHELF LIFE

- Use before expiry date.

VI. STORAGE

- CLEARFIL AP-X should be stored below 25°C (77°F) when not in use.
- Keep away from extreme heat or direct sunlight.

[CAUTION]

Federal (U.S.A.) law restricts this device to sale by or on the order of a licensed dentist.

[WARRANTY]

KURARAY CO., LTD. will replace any product that is proved to be defective. KURARAY CO., LTD. does not accept liability for any loss or damage, direct, consequential or special, arising out of the application or use of or the inability to use these products. Before using, the user shall determine the suitability of the products for the intended use and the user assumes all risk and liability whatsoever in connection therewith.

CLEARFIL is a trade mark of KURARAY CO., LTD.
VITA® is a trade mark of VITA Zahnfabrik, H.Rauter GmbH & CO KG Bad Säckingen, Germany.

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Las leyes federales (U.S.A.) limitan la venta de este producto a dentistas con licencia personas autorizadas por ellos.
Fabricado por**KURARAY CO., LTD.**