ENGLISH

Refer to the Instructions for use of CLEARFIL SE BOND prior to use.

I. Introduction

This bottle cartridge is designed to provide easy dispensing and to prevent spilled monomers from coming in contact with the skin.

II. Part names

Consult the illustration.

- (1) Outer case
- (2) Cap
- (3) Case clasp
- (4) Press points

III. Prior to use

1. Remove the outer case cap.
2. Push the case clasp and open the outer case.
3. Remove bottle caps of PRIMER and BOND, then set the bottles into the outer case.

IV. How to use the outer case

1. Remove the cap
Hold the outer case and remove the cap vertically upward.
Avoid pushing the press points or turning the cap.

[NOTE]

After taking out of the refrigerator, the product must be left standing until it comes to room temperature; otherwise the liquid might drop more than necessary or ooze after use.

[CAUTION]

Do not hold the press points when removing the cap; otherwise the liquid might spout out unexpectedly.

2. Dispensing PRIMER and BOND

- 1) Hold the bottle case and point it vertically downward.
- 2) Push the press point to dispense the necessary amount of PRIMER (orange liquid) into a well of the mixing dish.
- 3) Push the press point to dispense the necessary amount of BOND (light yellow liquid) into another well of the mixing dish.
- 4) Replace the cap as quickly as possible after dispensing the liquids.

[CAUTION]

Do not mix PRIMER with BOND.

When dispensing PRIMER or BOND, hold the bottle case and point it vertically downward. If the outer case is slanted, the liquid may adhere to the case when dispensing.

[NOTE]

Do not put the bottle case with it bottom up; otherwise the liquid might drop more than necessary.

3. Light blocking

Cover the mixing dish with the light blocking plate to avoid operating light or natural light (sunlight from windows). Within three minutes from dispensing, use PRIMER (orange liquid) and BOND (light yellow liquid) according to the application instructions.

[CAUTION]

PRIMER and BOND polymerize by visible light (especially ultraviolet rays). Use the light blocking plate to avoid operating light or natural light (sunlight from windows) and use within 3 minutes from dispensing.

[CAUTION]

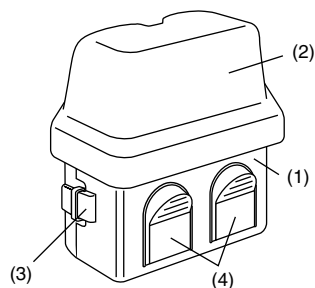
Federal (U.S.A) law restricts this device to sale by or on the order of a licensed dentist.

[WARRANTY]

KURARAY MEDICAL INC. will replace any product that is proved to be defective. KURARAY MEDICAL INC. does not accept liability for any loss or damage, direct, consequential or special, arising out of the application or use of or the inability to use these products. Before using, the user shall determine the suitability of the products for the intended use and the user assumes all risk and liability whatsoever in connection therewith.

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CLEARFIL is a trademark of KURARAY CO., LTD.
CLEARFIL es una marca registrada de KURARAY CO., LTD.