ENGLISH

Read the instructions for use of CLEARFIL PROTECT BOND prior to using.

I. Introduction

The bottle carrier is designed to provide easy dispensing and to prevent spilled monomers from coming in contact with the skin.

II. Component names

Refer to the illustration.

- (1) Outer case
- (2) Cap
- (3) Case clasp
- (4) Press points

III. Prior to use

- 1. Remove the outer case cap.
- 2. Press the case clasp and open the outer case.
- 3. Remove bottle caps from PRIMER and BOND, then place the bottles into the outer case.

IV. How to use the outer case

- 1. Remove the outer case cap
Hold the outer case and remove the cap vertically upward. Avoid pushing the press points or turning the cap.

[NOTE]

After taking out of the refrigerator, the product must be left standing until it comes to room temperature; otherwise the liquid might drop more than necessary or ooze after use.

[CAUTION]

Do not hold the press points when removing the cap; otherwise the liquid might spout out unexpectedly.

2. Dispensing PRIMER and BOND

- 1) Hold the bottle case and point it vertically downward.
- 2) Push the press point to dispense the necessary amount of PRIMER (light blue liquid) into a well of the mixing dish.
- 3) Push the press point to dispense the necessary amount of BOND (light yellow liquid) into another well of the mixing dish.
- 4) Replace the cap as quickly as possible after dispensing the liquids.

[CAUTION]

Do not mix PRIMER with BOND.
When dispensing PRIMER or BOND, hold the bottle case and point it vertically downward. If the outer case is slanted, the liquid may adhere to the case when dispensing.

[NOTE]

Do not store the bottle case with the bottom up; otherwise the liquid might drop more than necessary.

3. Light blocking

Cover the mixing dish with the light blocking plate to avoid exposing the material to an operating light or natural light (sunlight from windows). Within 3 minutes from dispensing, use PRIMER and BOND according to the application instructions.

[CAUTION]

PRIMER and BOND will set to a gel if left under an operating light or natural light (sunlight from windows). Use the light blocking plate to avoid exposing the material to an operating light or natural light, and use within 3 minutes from dispensing.

[CAUTION]

Federal (U.S.A) law restricts this device to sale by or on the order of a licensed dentist.

[WARRANTY]

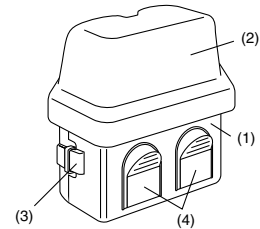
KURARAY MEDICAL INC. will replace any product that is proved to be defective. KURARAY MEDICAL INC. does not accept liability for any loss or damage, direct, consequential or special, arising out of the application or use of or the inability to use these products. Before using, the user shall determine the suitability of the products for the intended use and the user assumes all risk and liability whatsoever in connection therewith.

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CLEARFIL is a trademark of KURARAY CO., LTD.
CLEARFIL es una marca registrada de KURARAY CO., LTD.