# CLEARFIL™ AP-X PLT

## ENGLISH INSTRUCTIONS FOR USE

### Description

CLEARFIL AP-X PLT is a light-cured, radiopaque restorative composite resin which provides accurate color-matching, high polishability and excellent physical properties, making it ideal for both anterior and posterior restorations. It is formulated with optimal viscosity assuring easy handling and placement. CLEARFIL AP-X PLT, with the special dispensing system, can be quickly placed directly into the cavity.

CLEARFIL AP-X PLT will provide optimal results when used with one of the CLEARFIL bonding systems, such as CLEARFIL SE BOND or CLEARFIL LINER BOND 2V. Used in combination with CLEARFIL SE BOND or CLEARFIL LINER BOND 2V, CLEARFIL AP-X PLT will produce restorations with superb aesthetics and durability, high bond strength and less pulpal irritation.

### I. Indications

CLEARFIL AP-X PLT is indicated for the following restorative applications

1. Class I, II, V restorations of posterior teeth
2. Class III, IV, V restorations of anterior teeth
3. Cervical cavities or root surfaces defects

### II. Contraindications

Patients with allergies to methacrylate monomers.

### III. Incompatibility

Eugenol containing materials for pulp protection and temporary sealing should not be used since they retard the curing process.

### IV. Precautions

#### 1. Safety precautions

- Avoid use of the product for patients with a history of hypersensitivity to methacrylate monomers.
- If any hypersensitivity, such as a rash or dermatitis, occurs from with the product, discontinue the use of the product and consult a physician.
- Wear gloves or take other appropriate protective measures to prevent the occurrence of hypersensitivity that may result from contact with methacrylate monomers, acetone or acid.
- Use caution to prevent the product from coming in contact with the skin or getting into the eyes. Before using the product, cover the patient's eyes with a towel to protect the patient eyes from possible splashing material.
- If the product comes in contact with human body tissues, take the following actions:  
<If the product gets in the eyes>  
Immediately wash the eye with copious amounts of water and consult a physician.  
<If the product comes in contact with the skin>  
Immediately wipe the material off using a cotton pledget (or gauze) moistened with alcohol and then wash with copious amounts of water.
- Use caution to prevent the patient from accidentally swallowing the product.
- Avoid looking directly at the curing light when curing the product.
- Keep the product out of reach of children.

#### 2. Handling and manipulation precautions

- If necessary, use a rubberdam in order to prevent moisture contamination.
- Do not use CLEARFIL AP-X PLT in conjunction with the other composite resin. Mixing materials may cause a change in physical properties, possibly a decrease, from the properties expected.
- When light curing the material, note the light curing depth in this Instructions For Use.
- If necessary, light curing separately to cure the materials completely.

#### 3. Storage precautions

- Do not use after the expiration date. Note expiration date on the outside of package.
- Keep away from extreme heat or direct sunlight.
- CLEARFIL AP-X PLT should be stored below 25°C when not being used.

### V. System Shades and Components

#### 1. Shades

CLEARFIL AP-X PLT is available in 8 shades; 6 shades correspond on a best-match basis with VITA Shade Guide plus 2 shades for special needs. Select an appropriate shade by matching the shade guide to the restoration site. The XL and HO shades offer a strong, lighter white color. The HO shade is translucent white and is suitable for sites requiring extra white color.  
Standard shades: A2, A3, A3.5, B2, B3  
Cervical shades: A4  
Translucent shades: XL, HO

#### 2. Components

CLEARFIL AP-X PLT  
20 tips(0.2g/tip) : A2, A3, A3.5, A4, B2, B3, XL, HO  
Note: All 8 shades are available separately.

### VI. Clinical Procedures

#### 1. Shade selection

Teeth should be cleaned to remove plaque or stains. Before isolating the teeth, select an appropriate shade using the VITA Lumin VACUUM shade guide.

#### 2. Cavity preparation

CARIES DETECTOR is recommended as a guideline for safe and necessary caries removal. A rubber dam and cotton rolls are recommended to keep the tooth dry and clean.

#### 3. Pulp protection

Any actual or near pulp exposure should be covered with a hard setting calcium hydroxide material. However, usually a cement lining or basing is not necessary. Do not use eugenol materials for pulp protection.

#### 4. Application of bonding agent

Use of either CLEARFIL SE BOND or CLEARFIL LINER BOND 2V is especially recommended, although all CLEARFIL bonding agents can be used. Follow the Instructions for Use for the specific bonding agent chosen.

#### 5. Placement and light curing of CLEARFIL AP-X PLT

##### 1) Dispensing

Place a PLT(Pre-loaded tip) into the dispenser barrel according to the instructions for use for dispenser.  
(Centrix syringe such as CR EZ or MARK IIIP is recommended.)  
The tip may be rotated to gain the proper angle for entrance into the cavity.  
Squeeze the dispenser with a slow, steady pressure.  
Excessive force is not necessary. Remove and discard the tip after use.

##### 2) Placement

Incremental placement and light-curing each increment is strongly recommended especially in deep cavities and in Class II cavities.

##### 3) Curing

Cure the resin with a dental visible light curing unit. Hold the light tip as close to the resin as possible.

[NOTE]

Shades	curing depth (mm)	
	20sec.	40sec.
A2, A3, B2, XL, HO	2.0	2.5
A3.5, A4, B3	1.5	2.0

Use of light curing units conforming to ISO/TS 10650:1999 is recommended. If necessary, confirm the curing depth using method 7.10 of ISO 4049:2000.

#### 6. Finishing

Contour the restoration and adjust the occlusion and using a fine diamond point. Polish with silicon rubber points or polishing discs.

[CAUTION]

Federal (U.S.A) law restricts this device to sale by or on the order of a licensed dentist.

[WARRANTY]

KURARAY MEDICAL INC. will replace any product that is shown to be defective. KURARAY MEDICAL INC. does not accept liability for any loss or damage, direct, consequential or special, arising out of the application or use of or the inability to use these products.

Before using, the user shall determine the suitability of the products for the intended use and the user assumes all risk and liability whatsoever in connection therewith.

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CLEARFIL is a trade mark of KURARAY CO., LTD. VITA is a trade mark of VITA Zahnfabrik, H.Rauter GmbH & CO KG Bad Sackingen, Germany.
CLEARFIL es una marca registrada de KURARAY CO., LTD.
VITA es una marca registrada de VITA Zahnfabrik, H.Rauter GmbH & CO KG Bad Sackingen, Alemania.