CLEARFIL™ REPAIR Multi Purpose

ENGLISH INSTRUCTIONS FOR USE

I. INTRODUCTION

CLEARFIL REPAIR Multi Purpose consists of K-ETCHANT GEL, ALLOY PRIMER, CLEARFIL CERAMIC PRIMER, CLEARFIL ST OPAQUER, SURFACE COAT and accessories.

K-ETCHANT GEL is a 40% phosphoric acid aqueous solution etching gel.

ALLOY PRIMER is a metal adhesive primer and CLEARFIL CERAMIC PRIMER is a silane-coupling agent.

CLEARFIL ST OPAQUER is a light-cured restorative composite resin used to mask exposed metal surface and tooth discoloration. SURFACE COAT is light-cured resin glaze material that provides a clear and smooth surface.

The product (kit) is a repair system for prosthetic restorations fabricated from porcelain, ceramic, hybrid ceramics (ESTENIA C&B) or composite resin.

II. INDICATIONS

- Repairing fixed prosthetic restorations fabricated from porcelain, ceramic, hybrid ceramics or composite resin (e.g. inlays / onlays, crowns, bridges and the superstructure of implants).
- Re-glazing fixed intra-oral restorations (e.g. cured direct / indirect composite resin, provisionals, acrylic appliances, glass ionomers)

III. CONTRAINDICATION

The product components should not be used to treat patients with a history of hypersensitivity to methacrylate monomer or acetone. Patients with previous allergic history to these products may develop rash or dermatitis.

IV. POSSIBLE SIDE EFFECT

The mucous membrane may turn a whitish color when in contact with CLEARFIL CERAMIC PRIMER, due to the coagulation of proteins by the monomer. This is a temporary condition which will disappear in a short period of time.

V. INCOMPATIBILITY

Do not use materials containing eugenol with the kit; eugenol has been shown to retard the polymerization and curing process.

VI. PRECAUTIONS

1. Safety precautions

- Avoid using the kit on patients with a known history of hypersensitivity to methacrylate monomers or acetone.
- If the patient demonstrates a hypersensitivity reaction, such as rash, eczema, features of inflammation, ulcer, swelling, itching, or numbness, discontinue the use of the product and seek medical attention.
- Use caution when using the product components and prevent contact with the soft oral tissue or skin. If the product components come in contact with the soft oral tissue or skin, wipe away with a cotton pledget moistened with alcohol, then immediately rinse with copious amounts of water. Prior to use the product components, cover the patient's eyes with a towel to protect them from splashing materials. If the product gets in the eye, immediately rinse the eye with copious amounts of water and consult an ophthalmologist.
- If the patient begins to feel sick or ill after inhaling the acetone contained in ALLOY PRIMER, encourage the patient to breathe copious amounts of fresh air.
- SURFACE COAT contains a multi-functional methacrylate. It has been reported that, in a stimulation test using a product formulation containing multi-functional methacrylate applied to the mucous membranes of an animal, histopathological observation revealed the formation of blisters. Pain, itching, or the formation of blisters may occur if unpolymerized resin in this product adheres and remains in contact with the soft oral tissues. If unpolymerized resin comes in contact with soft oral tissue, wipe the material away with a cotton pledget moistened with alcohol and rinse with copious amounts of water. Any of the above symptoms may occur if unpolymerized resin remains on the tooth surface after light curing. To prevent the occurrence of these symptoms, light cure this product completely or remove any traces of unpolymerized resin remaining on the surface by using dry gauze.
- Use caution to prevent swallowing any of the product components.
- Do not use the same Disposable brush tip for different patients to prevent cross contamination. Discard the tip after use and sterilize the Brush tip handle after each patient.
- Avoid direct contact with the skin and soft tissue to prevent hypersensitivity. Wear protective gloves when using the product.
- Do not look directly at the light source. Protective glasses are recommended.

2. Handling and manipulation precautions [K-ETCHANT GEL]

- K-ETCHANT GEL may harden when allowed to stand for long periods of time. If it thickens, cap the bottle securely and shake vigorously until this product returns to its original state.
- If K-ETCHANT GEL comes in contact with your clothing, wash it with water.
- If the treated adherent surface is contaminated with saliva or blood, rinse with water, dry and re-treat again with this product.

[ALLOY PRIMER]

- If the treated adherent surface is contaminated with saliva or blood, clean it with a cotton pledget saturated with alcohol, wash, dry, and re-treat again with this product.

[CLEARFIL CERAMIC PRIMER]

- CLEARFIL CERAMIC PRIMER should be used as soon as practicable after dispensing. It contains ethanol, which is highly

volatile. As the solvent evaporates, the viscosity of this product increases, which can make application difficult.

- If CLEARFIL CERAMIC PRIMER accumulates on the nozzle or the container, remove it by using a piece of gauze or a cotton pledget saturated with alcohol.
- If the treated adherent surface is contaminated with saliva or blood, rinse with water, dry, clean the area with K-ETCHANT GEL, and re-treat again with this product.
- After this product has been applied on a prosthetic restoration and completely dried, restorative material should be applied as soon as possible.

[CLEARFIL ST OPAQUER]

- CLEARFIL ST OPAQUER should be used under conditions which result in proper polymerization.
- Poor adhesion may result if the intensity of the dental curing unit's light is too low. Check the service life of the lamp and periodically examine the emitting tip for contamination. Routinely check the light intensity using a radiometer.
- The emitting tip of the dental curing unit should be held vertical to the resin surface and held as close to the surface as possible.
- If the area to be light cured is larger than the bore of the emitting tip, divide the exposure process into intervals and light cure for the specified length of time.
- If this product is to be applied in a layer over 0.5 mm thick, or if it is to be applied in areas difficult to reach with the light, apply and cure this product in increments.
- After use, wipe remaining paste from the tip of the syringe before closing the cap.

[SURFACE COAT]

- During light curing, SURFACE COAT may produce a brief exothermic reaction which is minimized by applying in a thin layer.
- Apply this product in a thin layer. A thick layer may cause air bubbles in the hardened resin or a yellowish surface on the restoration.
- Roughen the restoration surface with a diamond point before applying this product. Failure to roughen the restoration surface could result in a weak bond between this product and the restoration, thereby resulting in poor durability.
- This product should be used under conditions that result in proper polymerization. Do not attempt to use the LED dental curing unit to light cure this product. This product does not contain a catalyst that reacts to the wavelength of the LED dental curing unit.
- Poor adhesion will result if the dental curing unit generates too low an intensity of light. Check the service life of the lamp and periodically examine the emitting tip for contamination. Routinely check the light intensity using a radiometer.
- The emitting tip of the dental curing unit should be held vertical to the resin surface and as close to the surface as possible. If the area to be light cured is larger than the bore of the emitting tip, divide the exposure process into increments and light cure for the specified length of time.
- The patient should be instructed to avoid sources of stains such as coffee, tea, and red wine for 3 days after treatment with this product.
- The surface smoothness of restorations coated with this product will not last long term. The treatment should be regarded as a temporary. It is recommended that the restoration is checked a few times during the next three months.

[Handling and manipulation precautions common to ALLOY PRIMER, CERAMIC PRIMER, SURFACE COAT]

- Do not use these products near an open flame. These products contain flammable substances.
 - Do not mix these products with bonding agents or use in conjunction with other products; combining materials may cause a change in physical properties, including a possible decrease in bond strength.
- [Handling and manipulation precautions common to all components]**
- Any exposed pulp or areas near the pulp should be covered with suitable material.
 - Provide moisture control with rubber dam, if necessary.
 - The product may only be used by a licensed dental professional.
 - Do not use the product for any other purposes other than those specified in INDICATIONS.

3. Storage precautions

- The product should be used before the expiration date indicated on the package.
- CLEARFIL CERAMIC PRIMER should be stored in a refrigerator (2 - 8°C / 36 - 46°F) when not being used. The other products should be stored at 2 - 25°C / 36 - 77°F. Do not expose the product to direct sunlight or store under an operating light.
- When the product has been stored in a refrigerator, it should be allowed to stand for more than 15 minutes and returned to room temperature before being dispensed. ALLOY PRIMER, CLEARFIL CERAMIC PRIMER and SURFACE COAT when used before the liquid returns to room temperature is subject to air bubbles which might get entrapped in the liquid and /or the material may ooze out of the cap after use, due to expansion.
- Cap the syringe and the bottle securely and as soon as possible after using the product. Failure to do so may cause evaporation of the ethanol, acetone, purified water, or methyl methacrylate contained in the product. CLEARFIL ST OPAQUER may also collect in the tip of the syringe, making it difficult to dispense.
- The product should be carefully stored and used by a licensed dental professional.

VII. COMPONENTS

1. Kit Components

- K-ETCHANT GEL : 6 ml
- ALLOY PRIMER : 5 ml
- CLEARFIL CERAMIC PRIMER : 4 ml
- CLEARFIL ST OPAQUER (Shade: US) : 4.0 g
- SURFACE COAT : 5 ml
- Accessories :
 - Disposable brush tips : 50 pièces
 - Disposable brush tips (flat type) : 20 pieces
 - Brush tip handle : 2 pièces (white, black)
 - Mixing dish : 1 piece
 - Light blocking plate : 1 piece
 - Paper pad : 1 piece

2. Ingredients

- Principal ingredients
- K-ETCHANT GEL
 - Water
 - Phosphoric acid
 - Colloidal silica
 - Dye
 - ALLOY PRIMER
 - Acetone
 - 10-Methacryloyloxydecyl dihydrogen phosphate (MDP)
 - 6-(4-Vinylbenzyl-N-propyl)amino-1,3,5-triazine-2,4-dithione

3) CLEARFIL CERAMIC PRIMER

- Ethanol
- 3-Methacryloxypropyl trimethoxysilane
- 10-Methacryloyloxydecyl dihydrogen phosphate (MDP)

4) CLEARFIL ST OPAQUER (Shade: US)

- Silanated silica
- Silanated colloidal silica
- Colloidal silica
- Bisphenol A diglycidylmethacrylate (Bis-GMA)
- Triethyleneglycol dimethacrylate (TEGDMA)
- d-Camphorquinone
- Initiator
- Accelerator
- Pigments

5) SURFACE COAT

- Methyl methacrylate
- Aliphatic hexaacrylate
- Initiator

VIII. CLINICAL PROCEDURES

Method. A

Repairing fixed prosthetic restorations fabricated from porcelain, ceramic, hybrid ceramics or composite resin (e.g. inlays / onlays, crowns, bridges and the superstructure of implants).

- Moisture control
 - Use rubber dam, if necessary, to provide moisture control.
- Roughening the adherent surface
 - Roughen the adherent surface with a diamond point, rinse and dry. If necessary, place a bevel at the marginal area.
- Etching the adherent surface
 - For a prosthetic restoration made of porcelain, ceramic, hybrid ceramics or composite resin, apply K-ETCHANT GEL on the surface, using the Disposable blush tip; leave in place for 5 seconds. Rinse and then dry. Zirconia-alumina prosthetic restorations do not require phosphoric acid etching.
- Conditioning exposed metal surfaces
 - If the adherent surface contains a metal surface, apply ALLOY PRIMER using the Disposable blush tip and dry it at room temperature.
- Treatment of adherent surfaces of porcelain, ceramic, hybrid ceramics or composite resin restoration
 - Apply CLEARFIL CERAMIC PRIMER to the adherent surface using the Disposable blush tip. Dry the liquid sufficiently using a low or medium pressure stream of air.
- Bonding treatment on exposed tooth surfaces
 - If the adherent surface includes tooth structure, treat the area with a bonding agent (e.g. CLEARFIL TRI-S BOND, CLEARFIL PROTECT BOND or CLEARFIL SE BOND) according to the product Instructions for Use.
- Masking metallic surface
 - If the adherent surface contains a metal surface, apply a thin layer of CLEARFIL ST OPAQUER, 0.5 mm or less, and then light cure the paste layer.
 - See the following tables for details regarding the types of the dental curing units, the wavelengths of light sources and the recommended curing times for CLEARFIL ST OPAQUER.

Table 1: Dental curing unit

Type	Light source	Wavelength range and light intensity
Conventional halogen	Halogen lamp	Light intensity ²⁾ of 300 - 550 mW/cm ² in wavelength range from 400 - 515 nm
Fast halogen	Halogen lamp	Light intensity ²⁾ of more than 550 mW/cm ² in wavelength range from 400 - 515 nm
Plasma arc	Xenon lamp	Light intensity ³⁾ of more than 2000 mW/cm ² in wavelength range from 400 - 515 nm, and light intensity of more than 450 mW/cm ² in wavelength range from 400 - 430 nm
LED	Blue LED ¹⁾	Light intensity ²⁾ of more than 300 mW/cm ² in wavelength range from 400 - 515 nm

- Peak of emission spectrum: 450 - 480 nm
- Evaluated according to ISO 10650-1
- Wavelength distribution and light intensity values measured with a spectroradiometer calibrated using IEC or the NIST (National Institute of Standards and Technology) standard lamp.

Table 2: Relationship between dental curing unit and curing time for CLEARFIL ST OPAQUER

Dental curing unit	Thickness of CLEARFIL ST OPAQUER	
	less than 0.4 mm	0.4 - 0.5 mm
Conventional halogen	40 sec.	60 sec.
Fast halogen	10 sec.	20 sec.
Plasma arc	5 sec.	5 sec.
LED	40 sec.	80 sec.

- Loading and light curing dental filling composite resin
 - Select the shade of composite resin (e.g. CLEARFIL MAJESTY Esthetic or CLEARFIL MAJESTY Flow) that matches the color of the tooth to be restored. Place and light cure the selected composite resin paste according to the Instructions for Use.

- Finishing / polishing or glazing by SUFACE COAT. Choose 9-a or 9-b based on if SURFACE COAT will be used.

- 9-a. Finishing / polishing of restoration
 - After light curing the composite resin, contour, finish and polish the polymerized resin as usual.

- 9-b. Using SURFACE COAT

- (1) Contour and roughen the restoration surface using diamond point, rinse and dry.
- (2) Dispense the necessary amount of SURFACE COAT into a well of the Mixing dish immediately before application.
- (3) Apply a thin layer of this product to the restoration surface with the Disposable blush tip (flat type). This flat type is recommended for this purpose.
- (4) Light cure this product using the dental curing light. Use the curing time shown in the table 3.

[NOTE]

Check for any unpolymerized resin on the cured SURFACE COAT using an explorer. If an unpolymerized resin layer is found, light cure again or remove with dry gauze.

Table 3: Relationship between dental curing unit and curing time for SURFACE COAT

Dental curing unit	Curing time
Conventional halogen	20 sec
Fast halogen & Plasma arc	10 sec

Method. B

Re-glazing fixed intra-oral restorations (e.g. cured direct / indirect composite resin, provisionals, acrylic appliances, glass ionomers)

- Roughening the surface of the prosthetic restoration
 - Roughen the surface of the prosthetic restoration you want to re-glaze using a diamond point; rinse sufficiently and dry.
- Applying SURFACE COAT
 - For details on application for SURFACE COAT, see Method. A 9-b.

[CAUTION]

Federal (U.S.A.) law restricts this device to sale by or on the order of a licensed dentist.

[WARRANTY]

KURARAY MEDICAL INC. will replace any product that is proved to be defective. KURARAY MEDICAL INC. does not accept liability for any loss or damage, direct, consequential or special, arising out of the application or use of or the inability to use these products. Before using, the user shall determine the suitability of the products for the intended use and the user assumes all risk and liability whatsoever in connection therewith.

[NOTE]

ESTENIA, SURFACE COAT, CLEARFIL ST, CLEARFIL TRI-S BOND, CLEARFIL SE BOND, CLEARFIL PROTECT BOND, CLEARFIL MAJESTY and CLEARFIL are trademarks of KURARAY CO., LTD.

[ALLOY PRIMER]

Manufactured by

KURARAY MEDICAL INC.
1621 Sakazu, Kurashiki, Okayama, 710-0801, Japan

Distributed by
KURARAY AMERICA, INC.
600 Lexington Avenue, 26th Floor, New York, NY10022
Tel. (800) 879-1676
Fax. (888) 700-5200
www.kuraraydental.com

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La ley federal (U.S.A.) restringe la venta de este dispositivo a dentistas autorizados o por orden de los mismos.